## BioGlo Ophthalmic Strip

### DESCRIPTION

BioGlo strips have an overall length of 60mm. The holding paper is 7mm in width. The impregnated portion is sandwiched between the holding paper and has an exposed portion of 15 mm by 5mm. The end of the tip rounded with a radius of 2.5 mm.

The absorbent paper holding the dye is made of Whatman # 1 or its equivalent. The holding paper is white in color, non-shredding and sufficiently stiff to offer a convenient hold.

Each strip is individually packed and sealed in a non-coated medical grade paper and film pouch to render a sterile product. The gsm of medical grade paper is not less than 60 gsm.

### INDICATIONS

Intended for use when fitting contact lenses.

Each strip is impregnated with 1 mg of fluorescein sodium. The strip is made of an absorbent paper, which is impregnated with the dye. This portion has a smoothly rounded tip that offers comfort during use. The absorbent portion is sandwiched between stiff, non-absorbent holding papers. The holding paper offers rigidity to the strip which makes it easy and convenient for use.

### CONTRAINDICATIONS

BioGlo strips are not meant for use with soft contact lenses as these will stain the lenses.

### WARNING

- Keep out of reach of children
- For external use only
- Discontinue use in case of persistent irritation or burning sensation
- Do not use if the pouch is torn, damaged or shows traces of leakage
- For professional use only

### STORAGE

- Store at room temperature 5 - 55 degree Celcius (41 - 131 degree Fahrenheit)
- Keep dry, do not store in moist environment

### DIRECTIONS

Handle the exposed strip by grasping the white holding paper. Moisten the impregnated area of the strip with sterile normal saline, or tear fluid from the lower fornix. While the patient looks up, stroke the moistened tip along the bulba conjunctiva. The patient should blink several times to obtain best results.

### HOW SUPPLIED

In boxes of 100 and 300 sterile strips.

NDC # 17238-900-11 (100 ct.)

NDC # 17238-900-30 (300 ct.)

### DISCLAIMER OF LIABILITY/RETURNS

The firm is liable only for potential manufacturing defect. Return the damaged container in its original packing identified by the lot number, your references and reason for return.

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Madhu Instruments Pvt. Ltd.
HUB Pharmaceuticals LLC
Med Devices Lifesciences Limited